# SleepDoctor®1.0

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# SleepDoctor® manual English

Read these instructions for use carefully, keep them for later use, make them available to other users and follow the instructions.

Check the scope of delivery for external intactness and completeness. Before use, make sure that no damage is visible and that the packaging material has been removed. If in doubt or if there is visible damage, please contact customer service.

## **1. Table of Contents:**

2. Scope of delivery
3. Device description
4. Commissioning
5. Use of the SleepDoctor® Consumer App
6. Safety and warning notices
7. Battery: maintenance, care
8. Technical data
9. Disposal
10. Orders
11. Contact

## **2. Scope of delivery**

Check the scope of delivery for external intactness and completeness. Before use, make sure that no damage is visible and that the packaging material has been removed. If in doubt or if there is visible damage, contact customer service.

- a) SleepDoctor® device

The scope of delivery consists of a SleepDoctor® packaging box with the following essential contents:

- A SleepDoctor® device including an integrated battery
- A chest strap

- A micro USB charging cable

b)


The associated SleepDoctor® Consumer App (IOS, Android) and updates, which allows you to further evaluate your sleep behaviour, is available from the Google Play Store, Apple Appstore or [www.sleepdoctor.de](http://www.sleepdoctor.de) :

<https://www.apple.com/de/app-store/> <https://play.google.com/store/apps/details?id=de.iptasys.sleepdoctor>

### 3. Description device

The following figures show a schematic representation of the device.

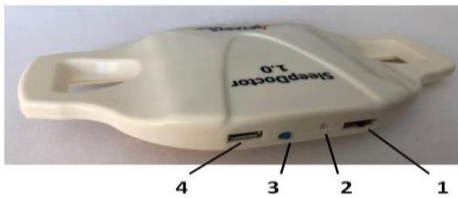1. on / off switch
- 2 On / Off test LED
- 3 Evaluation button
- 4 USB charging cable connection
- 5 chest strap connection point
- 6 top of the device

Do not dispose of with household waste. Disposal in accordance with waste electrical and electronic equipment, EC WEEE directive.

This product meets the requirements of the applicable European and national guidelines.

#### 4. Installation

**Remove all packaging materials and check the device for damage or defects. The device must not be used if it is damaged or faulty. Contact customer service.**

**Before using the device for the first time, it should be fully charged once (further information on this can be found below).**

**Switch the device on (switch (1) to position „On“). The test LED (2) lights up green for approx. 2 seconds.**

If the light is green, the battery will run for at least 8 hours. The device is now ready for use.

Attach the chest strap supplied on the connection points (5) on the device. Change the length of the chest strap as desired. Put on the SleepDoctor®. The device should be worn close to the chest above the breastbone. Women should wear the SleepDoctor under the female breast. Alternatively, it can also be worn over the female breast, but it must then be ensured that the SleepDoctor fits tightly on the chest, otherwise the device cannot work reliably.

With the SleepDoctor on and switched on, you can go to sleep with it. The SleepDoctor then monitors your sleep.

The SleepDoctor records data about your sleep: sleeping position, sleep phases, snoring, respiration.

The device detects snoring in any sleeping position and vibrates as soon as someone snores for longer.

The device vibrates if you have not breathed for more than 10 seconds.

After the device has vibrated, it will vibrate again after 10 minutes at the earliest so that the sleep rhythm is not impaired.

Please note: taking medication (e.g. psychotropic drugs or others), consuming alcohol, loosening the device during sleep can impair the measuring function of the device or the person's reaction to the vibration.

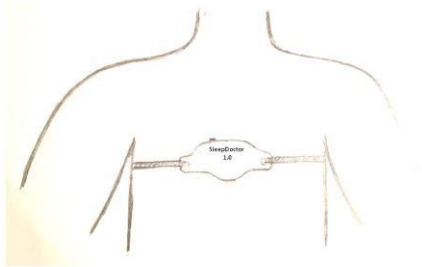
In the morning, open the chest strap to remove the device.

If you switch off the device (without synchronization with the SleepDoctorApp), the sleep data saved from the previous night will be deleted. So do not switch off the SleepDoctor before you have synchronized your data with your electronic device.

The app for data synchronization can be downloaded from the Google Play Store (SleepDoctor app) or [www.SleepDoctor.de](http://www.SleepDoctor.de). The operating instructions can be accessed electronically there too. The data from the SleepDoctor device is synchronized with your smartphone or tablet using the small blue button (see operating instructions, no. 3 "Evaluation"). To transfer data from your SleepDoctor to your electronic device, press and hold the blue evaluation button until you can see your data on your device. The SleepDoctor must be no more than 50 cm away from your electronic device.

**If the battery is empty, the LED lights up red after switching on the device and must be charged. The device should be charged daily.**

**You can switch the device off again by pressing the on / off switch (1).**

## 5. Use of the SleepDoctor® Consumer App

Synchronization: data transfer and evaluation of sleep data on the smartphone / tablet

To receive further evaluations of your sleep behaviour using the SleepDoctor® Consumer App, synchronize the SleepDoctor® device with your smartphone after waking up. **Important: after waking up, do not switch off the device before the data has been transferred!**



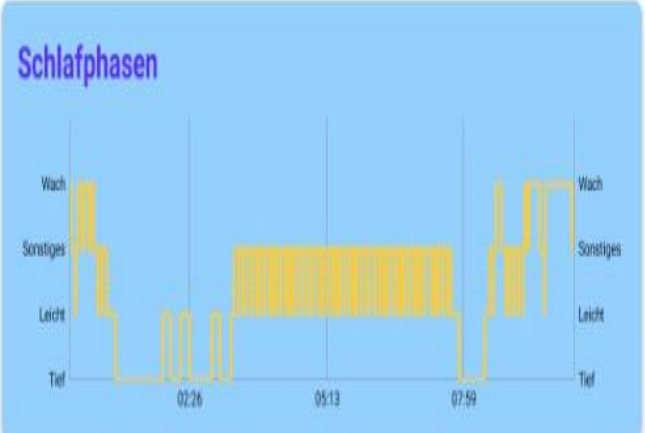
Carry out the following steps:


- Taking off the SleepDoctor® device after sleeping
- Start the SleepDoctor® Consumer App on your smartphone
- Select the “Start Sync” function under “Synchronize”. The app is now ready to sync.
- Press the blue evaluation button (3) on your SleepDoctor® device to start the synchronization. The synchronization is completed after a few seconds.

**Evaluations:**

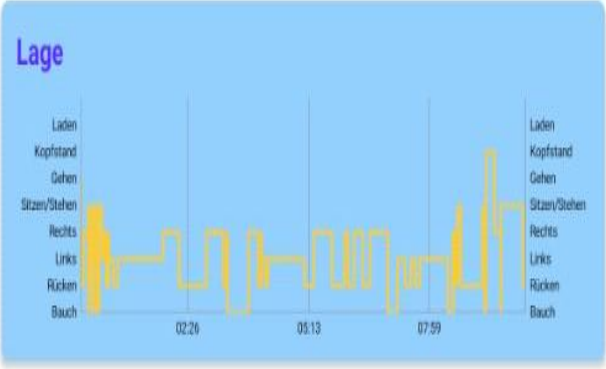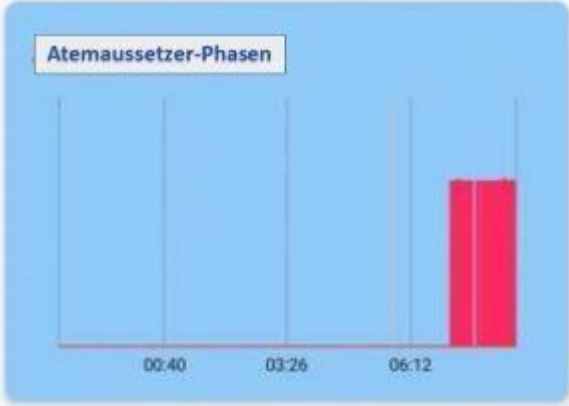The following evaluations are available in the SleepDoctor® Consumer App:

Evaluation criterion	Example	Notes
Sleep quality		<p>This gives you a summary of your sleep quality. The longer the deep sleep phases and the lower the proportion of snoring and breathlessness, the better your sleep quality. Bad sleep quality means less than 10%, good sleep 10-25%, very good sleep: more than 25%.</p>
Sleep phases		<p>Here you get an overview of your sleep phases from last night. The device differentiates between waking, light and deep sleep phases. Phases that are not clearly recognizable, including the REM sleep phase, are shown as "Other".  Wach: awake  Sonstiges: other  Leicht: light  Tief: deep</p>

<p>Position</p>	 <p>The graph titled "Lage" displays a yellow line representing the user's sleeping position over time. The y-axis lists positions: Laden, Kopfstand, Gehen, Sitzen/Stehen, Rechts, Links, Rücken, and Bauch. The x-axis shows time markers at 02:26, 05:13, and 07:59. The line fluctuates between several positions throughout the night, indicating a restless sleep.</p>	<p>In this evaluation you can see how often you have changed your sleeping position during the night. As a rule, the frequency with which you change position can be used to determine whether your sleep was rather calm or restless. Getting up at night is also recorded here.</p>
<p>Respiration/breathing</p>	 <p>The graph titled "Atemaussetzer-Phasen" shows respiratory arrest phases as red vertical bars. The x-axis has time markers at 00:40, 03:26, and 06:12. Two prominent red bars are visible after 06:12, indicating prolonged respiratory arrest.</p>	<p>Here you can see whether and when the SleepDoctor® has identified prolonged respiratory arrest during your sleep. If you regularly notice periods of prolonged respiratory arrest, it is advisable to consult a doctor.</p>
<p>Snoring</p>	 <p>The graph titled "Schnarchen" shows snoring events as purple vertical bars. The x-axis has time markers at 02:59, 06:46, and 10:33. Two purple bars are visible, one around 06:46 and another near the end of the night, representing snoring events.</p>	<p>How often do you snore? This evaluation shows you all snoring events recorded during the night including their duration.</p>

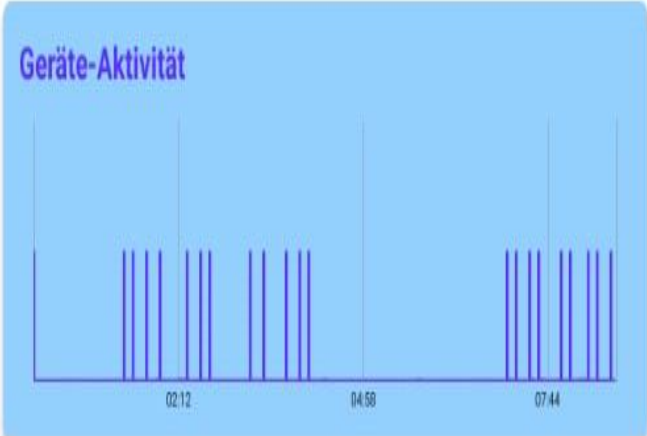Device activity		The device activity shows you how often the SleepDoctor® reacted to sleep events such as snoring with a vibration during your sleep.
-----------------	--	--

The SleepDoctor App will soon be available in English. Please check regularly for updates in Google Play Store and Apple Store.

## 6. Safety and warning notice

THE DEVICE IS FOR DOMESTIC / PRIVATE USE ONLY AND NOT

INTENDED FOR MEDICAL OR COMMERCIAL USE.

THE DEVICE IS NOT WATERPROOF; PLEASE PROTECT IT FROM SPLASHING.

DO NOT DROP THE DEVICE, THE CASE AND THE ELECTRONICS COULD BE DAMAGED.

STORE AND USE THE DEVICE IN THE TEMPERATURE RANGE FROM 0 TO 40 ° C.

IF THE DEVICE IS OPENED, THE MANUFACTURER'S GARANTEE IS VOID.

## 7. Charging the battery, care, maintenance, repair:

The SleepDoctor® has a battery that can work for several nights. If the battery is empty, it can be recharged. The device can be charged with the enclosed infrastructure via a USB power supply unit. Before using the device for the first time, it should be fully charged. While the battery is charging, the LED flashes green. If the LED lights green continuously, the device is charged.

### General information on handling batteries - Warning !!

- Only use undamaged batteries. Batteries must not be opened, dismantled or divided.
- Batteries must be protected from excessive heat.
- RISK OF EXPLOSION! Do not overheat or burn.
- Ensure proper charging.
- Only use the devices listed in our instructions

## 8. Technical data:

Dimensions	138 x 76 x 13 mm		Battery runtime	Few nights
Weight	50g		Storage	Few hours
Connections	Micro-USB		Material	Polyamid
Power supply adapter	input: 100-240V, 50/60Hz, 0,3A output: 5V, 1000mA		Permitted operating conditions	0°C bis 40°C 30 bis 75% RH
Battery pack	800mAh NiMH		Permitted transport / storage conditions	0°C bis 40°C 30 bis 75% RH

## 9. Disposal

This symbol means that, in the interests of protecting the environment, the device will not be disposed of with household waste at the end of its service life.

Disposal can take place at appropriate collection points in your country. Dispose of the device in accordance with the waste electrical and electronic equipment EC directive - WEEE (Waste Electrical and Electronic Equipment).

Contact the authority responsible for disposal if you have any questions.

## 10. Order:

[www.sleepdoctor.de](http://www.sleepdoctor.de) [info@iptasys.de](mailto:info@iptasys.de)

## 11. Contact:

Iptasys GmbH

Rehweg 7

89081 Ulm

Tel: 004915224904533

Ulm District Court

HRB 724482

Managing Director: Dr.-Ing. Guy Kouemou Sales

tax identification number: DE269173623

[www.iptasys.de](http://www.iptasys.de)

[info@iptasys.de](mailto:info@iptasys.de)